
Specimen Packaging and Courier Services

Ambient Specimens

Our containers are designed to transport serum and urine specimens that do not require special temperatures or handling. If you have any concerns regarding the effect of extreme weather conditions on routine or refrigerated specimens, please call our administrative personnel.

Courier Services

Free specimen pickup service is available throughout our entire service area. Our couriers pick up specimens, provide supplies, and deliver reports on a schedule which closely meets your needs. Schedules are arranged in accordance with client needs. We think you will find that our service is of high quality, and our personnel are both reliable and responsive to your service needs.

Frozen Specimens

Send each frozen specimen in a vial not more than three-fourths full. Under no circumstances should you ship frozen specimens to us using coolants.

Refrigerated (On Coolant) Specimens

Specimens need to be kept refrigerated until courier pickup.
Note: OSHA requires that all shipments containing clinical specimens be marked with a "Biohazard Label." Bags and labels for shipments sent to CAMC LabWorks will be provided.